

Installation Instructions

Before you start downloading please make sure that you have administrative rights on your computer.

- Depending on your connection speed, the download may take up to a few minutes.
- When the download has finished, click **Run** to launch the Clearblue Cycle Calendar Widget.
- Follow the steps shown in the setup wizard by clicking **NEXT**. The installer will guide you through the steps required to install Clearblue Cycle Calendar on your computer.
- Select installation folder by clicking **NEXT**.
- Confirm installation by clicking **NEXT** again.
- Wait a few seconds until the installation process is complete than click **CLOSE** to exit.
- Look for the Clearblue icon on your desktop and with a double-click **START** the calendar minimized in the system tray (taskbar – next to the clock). In order to open and use it perform a single click on the system tray icon.

