


Clearblue®

Pregnancy Test

FLIP & CLICK

 If testing early use first urine of the day. Otherwise test any time

 Don't drink lots of liquid before testing

Over 99% accurate from the day you expect your period!

Read this leaflet carefully before testing.


1 Open handle fully and remove cap

Remove from foil and use immediately after opening.



2 Choose your testing method

Place the color change tip for **5 seconds**:

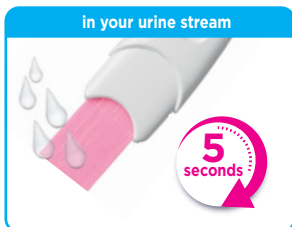

or



IMPORTANT: The color change tip will turn instantly pink showing that urine is being absorbed **but you MUST continue to hold the tip in your urine for a full 5 seconds**. Replace the cap and lay the test stick flat. If the tip does not turn completely pink, see 'Test errors'.


3 Wait



Blue lines will start to develop. If not, see 'Test errors'. **Wait 3 FULL minutes** before reading your result.

4 Results

Pregnant - if your result is pregnant see your doctor.

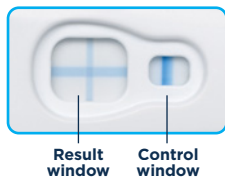

Result window Control window



Not Pregnant (see Q4)



A blue line must be present in the control window for your result to be valid. If not, see 'Test errors'.

IMPORTANT! Read your result within **10 minutes** of testing. Disregard any changes after this time.

? Questions and answers

1 How does Clearblue work?

It detects the pregnancy hormone hCG (human Chorionic Gonadotrophin) in urine. Test sensitivity is 25mIU/ml. It's over 99% accurate in laboratory tests from the day you expect your period¹.

2 How early can I test?

You can test from 5 days before your missed period (4 days before you expect your period). If you test before your expected period and get a 'Not Pregnant' result, there is still a chance that you may be pregnant. See the chart below for results of lab testing with early pregnancy samples.

Days before the expected period	-1	-2	-3	-4
Pregnancies detected	98%	97%	88%	56%

3 Can any medication or medical conditions affect the result?

Always read the manufacturers' instructions for any medication you are taking before testing. If you get unexpected results you should discuss them with your doctor. Medications containing hCG (e.g. Pregnyl^{®2} and Profasi^{®3}) can affect the result. Ectopic or recent pregnancy, even if not carried to full term, can cause misleading results.

The test should **NOT** be affected by hormone therapies containing Clomid^{®4}, common painkillers, alcohol, antibiotics, or the contraceptive pill.

4 My test says I'm 'Not Pregnant'. What should I do?

If your result is negative but you still think you may be pregnant, test again in 3 days' time. If this result is also negative but you still suspect pregnancy, see your doctor.

! Test errors

The color change tip did not turn completely pink.

You may not have applied enough urine. However, you can be sure the test has worked if the control line appears within 10 minutes of testing.

No blue line appears in the control window within 10 minutes of testing. The test has not worked, possibly because too much or too little urine was used. Test again using a new test, following the instructions carefully.



Result window Control window



www.clearblue.com

Clearblue® Helpline

Monday-Friday 8:30 a.m. - 5:00 p.m. Eastern Time.

Toll-free. 1-800-321-3279 Please have the product and packaging, including any foil pouches, with you when you call. Calls are recorded for training and quality control.

Home Pregnancy Test. For *in vitro* diagnostic use only.

Store at 36°-86°F (2°-30°C). Allow to reach room temperature for 30 minutes prior to testing if stored refrigerated. Do not freeze. Keep out of reach of children.

Do not use if the foil wrapper containing the test stick is damaged.

Do not use if the test stick is past its expiration date. Use once.

Distributed by Procter & Gamble, Cincinnati, OH 45202.

Clearblue is a trademark of SPD. © 2019 SPD. All rights reserved.

Pat. - see www.swissprecisiondiagnostics.com/patents

¹99% accurate at detecting typical pregnancy hormone levels.

Note that hormone levels may vary.

²Pregnyl[®] is a registered trademark of ORGANON USA, Inc.

³Profasi[®] is a registered trademark of Ares Trading S.A.

⁴Clomid[®] is a registered trademark of Sanofi-Aventis.

The US Surgeon General advises pregnant women to abstain from alcohol consumption as this may lead to harmful effects on the baby such as Fetal Alcohol Syndrome Disorders (FASD). More information is available at www.surgeongeneral.gov.